



## COVID-19 IgM/IgG Rapid Test

COVID-19 IgM/IgG Rapid Test is an *invitro* diagnostic test for the qualitative determination of COVID-19's IgM and IgG antibodies in human whole blood (from vein or fingertip), serum or plasma.

**Qualitative Fast Reliable**




### Steps for a single test

1. Add 10  $\mu$ L specimen
2. Add 2 drops of buffer
3. Read the test result

Note: please make sure read the result at 15 minutes. If at other time, it may cause wrong results

### Interpretation of results

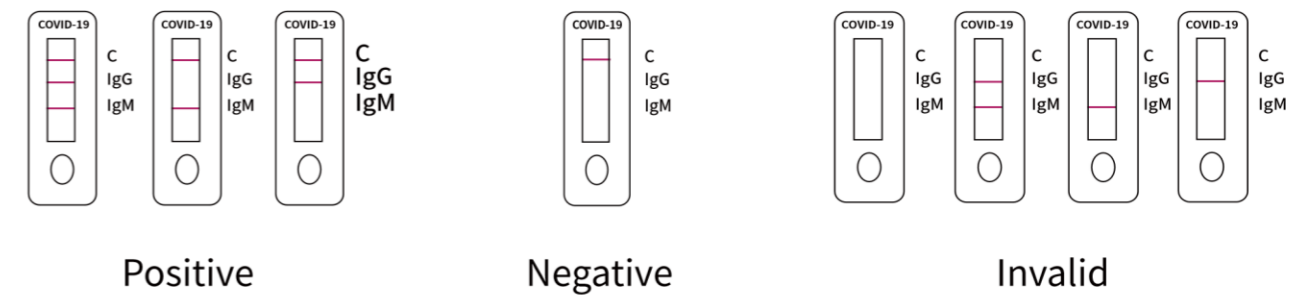

### Specification

Product Name	COVID-19 IgM/IgG Rapid Test
Test Principle	Colloidal gold
Sample Type	Whole blood (from vein or fingertip), serum or plasma
Sample Volume	10 $\mu$ L
Test Time	15 min
Operation Temperature	18-25°C
Storage Temperature	2-30°C
Shelf Life (Unopened)	12 months